




User Guide

Download the **AyuShare Mobile Application**



Scan the QR Code:

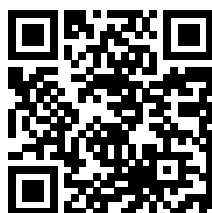


Download the app from Website



1.

AyuSynk Configuration

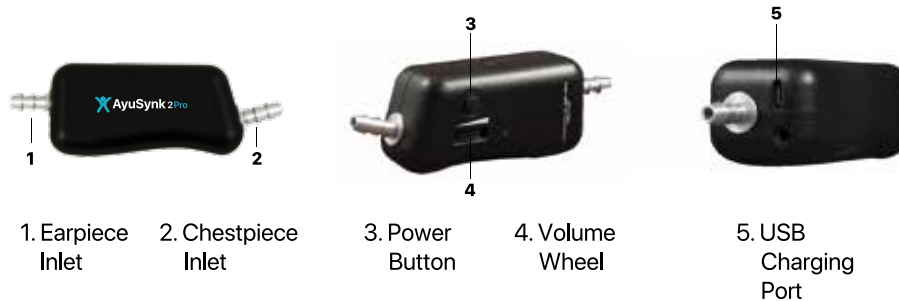

2.

How to attach AyuSynk to a Regular Stethoscope


1 Take your Regular Stethoscope




2 Detach the Chestpiece from the stethoscope



3 Connect the AyuSynk module to the tube

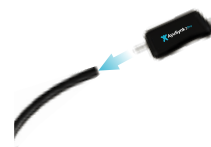

4 Select the required connecting tube and to the other end of AyuSynk attach it

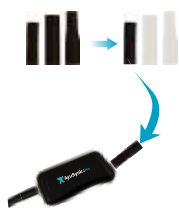

5 Connect the chest piece to the new extension




6 Your AyuSynk enabled digital stethoscope is ready to use



3.

How to Connect AyuSynk to a Chest Piece

- 1 Take your AyuSynk Module
- 2 Pick the smallest of the three connecting tubes given to and connect it towards the right end of AyuSynk
- 3 Connect the Chest piece to the new extension tightly

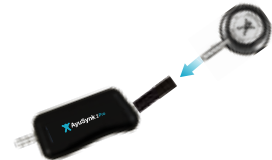

- 4 Your AyuSynk is now ready to use
- 5 Once ON, it will show a blinking Pink-white light indicating heartmode in default.



How to hold the device during Auscultation

Hold the Chest piece between your index and middle finger

Place your thumb on the volume button



4.

LED Indicators for AyuSynk

Device Status

Colour	Indication
● ○	Device ON / heart mode default
●	Low Battery

During Charging

Colour	Indication
●	Charging
●	Charging Full

Filter mode

Colour	Indication
● ○	Pink-white / heart mode
● ○	Blue-white / Lung mode
●	Cyan / No filter

5.

Connecting AyuSynk
with the App

1



○ First make sure your Ayu synk is showing a Pink-white blinking light.
●
○


2 Next, launch the AyuShare app on your phone



3 Click on "Allow" when asked for permission



4 At the Login Page, enter your credentials in Email Id Password or click on Sign in with google

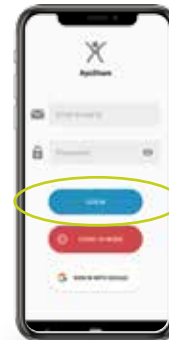

5 After Signing In you are taken to the home screen of the app here tap on "connect"



6 In the next step, in the "choose your device" screen select second option, "AyuSynk" -Connect via Bluetooth and click on done



7 Next, your device will be detected by your phone. Click on "Connect"



8 That's it you have connected your device to your phone. Now select any of the options and get started





Consulted the Best to **Deliver you the Best**

We have been extremely active during this global pandemic to continuously improve and craft out AyuLynk

Facing any issues?

Contact us at:

Purchase and sales Enquiry


+91 8047191146

After sales support

+91 8047191855

contact@ayudevices.com

www.ayudevices.com

 **Temperature
Limit (0 to 45°C)**

 **Atmospheric pressure
limitation**

 **Keep Dry**

Manufactured by:

Ayu Devices Pvt. Ltd.
3rd Floor, SINE, CSRE Building,
IIT Bombay, Powai, Maharashtra -400 076

**For More Video
Instructions
scan the QR Code**

